

# INTRODUCING

## SaliMark™ OSCC

The Most Clinically-Advanced and Scientifically-Validated Molecular DNA Test for Oral Squamous Cell Carcinoma



**SaliMark™ OSCC is the first in a series of advanced, accurate and easy-to-use salivary DNA tests in the robust product pipeline of PeriRx LLC, a premier developer of breakthrough, molecular diagnostic technology.**

### Oral Cancer: The Grim Statistics

*According to the National Institutes of Health, oral cancer is the sixth most common cancer in the world.*


- Over 43,000 people in the US will be diagnosed with oral cancer this year
- There are over 8,000 deaths each year in the US caused by oral cancer

### Early Detection Saves Lives!

- When diagnosed in later stages, the oral cancer survival rate is 50%
- When detected early, the survival rate for oral cancer is over 90%
- One person dies of oral cancer in the US every hour of every day

### SaliMark™ OSCC's Advanced Technology

- Rigorous clinical validation including independent studies by the National Cancer Institute
- The first risk assay of its kind to aid in the identification and early detection of oral lesions that present the highest risks for cancer
- Removes the guess work from the assessment of oral lesions
- Accurate, affordable and easy-to-use

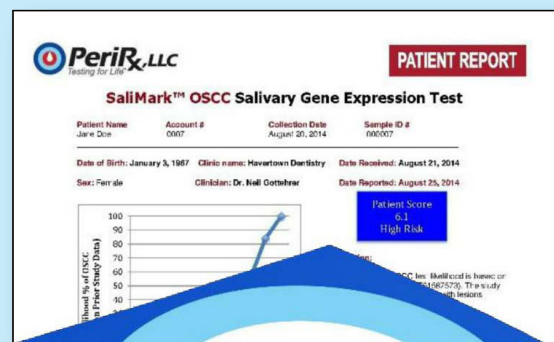

## How to Use the SaliMark™ OSCC Test




- Perform a thorough oral exam
- Oral exam identifies a potential suspicious lesion
- Discuss with the patient the role of the salivary test to quantitate risk
- Have the patient rinse their mouth
- Collect 2 cc of saliva (not sputum) in the specialized collection tube
- Identify the sample with the provided bar coded label including the patient name and birthday
- Register the patient in your account by name, birthday and unique ID number provided on the bar coded requisition

- Seal and ship by FedEx to the laboratory for analysis
- Log into the HIPAA secure web site to obtain results within 72 hours
- Patient referral guidelines are listed below

## How to Read the SaliMark™ OSCC Test Score

- A low-risk score is associated with a significantly reduced chance that an oral lesion is malignant and the clinician should follow up with the patient as appropriate
- A moderate-risk score may raise enough concern about an oral lesion to warrant referral to a specialist
- A high-risk score is associated with a significantly increased chance that an oral lesion is malignant, and referral to a specialist is recommended
- These results substantially enhance the clinical assessment of patients with oral lesions that are suspicious for cancer

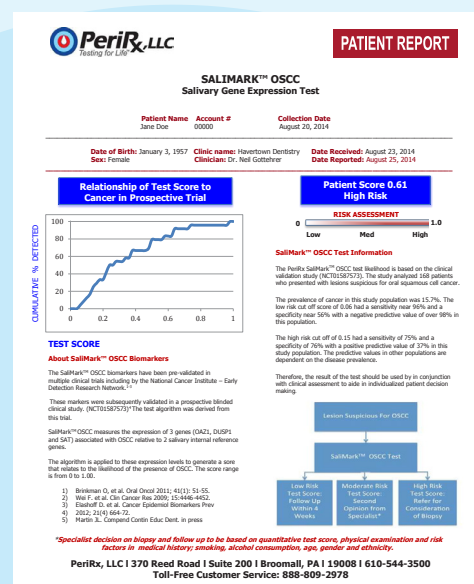

## About PeriRx, LLC

PeriRx LLC is a premier developer of breakthrough, non-invasive, oral diagnostic technology that will help clinical professionals detect and treat diseases sooner and enhance the practice of wellness management.

## Contact Us

**PeriRx, LLC**  
370 Reed Road | Suite 200 | Broomall, PA | 19008  
Tel: 610-544-3500 | Fax: 610-544-3510  
Info@PeriRx.com | www.PeriRx.com